Medc Medical

Today's innovation for a safer Tomorrow

ECOSAN NM-5
Advanced Disinfection Technology

Proven Efficacy
Up to 99.99%

Enhanced with
Nano Silver Technology

Environmentally Friendly
Safe and Biodegradable
Formula

Broad-Spectrum Efficacy
Against Bacteria, Viruses,
Fungi and Algae

Certified Quality
Tested According to
International Standards

Designed for Disinfection of
Surgical Instruments,
Medical Equipment and
Various Surfaces

Safe and Reliable
Advanced Formula for
Healthcare Environments

Approved by the Saudi
Ministry of Health and
Saudi Food and Drug Authority

1 INTRODUCTION

Ecosan NM-5 is an advanced national product representing a new generation in high-level disinfection and sterilization. Its formulation is based on **5% hydrogen peroxide** enhanced with **nano-silver technology**, providing broad-spectrum efficacy against microorganisms, including bacteria, viruses, coronaviruses, fungi, and algae.

The product is designed for the disinfection of surgical instruments, medical equipment, tissues, and various surfaces such as walls and floors, with proven effectiveness reaching up to **%99.99**.

Ecosan NM-5 has been evaluated and approved by the Saudi Ministry of Health and the Saudi Food and Drug Authority. It has also undergone rigorous testing in international laboratories in accordance with European and American standards, ensuring its reliability in advanced healthcare environments.

2 TECHNICAL SPECIFICATIONS

Contains hydrogen peroxide at a concentration of **5%**, enhanced with **nano-silver technology** to ensure maximum disinfection efficiency.

3 KEY FEATURES AND BENEFITS

Ready-to-use medical-grade disinfectant solution with high efficacy.

Advanced and safe formulation, free from silver nitrates, alcohol, aldehydes, chlorine, phenols, and quaternary ammonium compounds.

Easy to use, leaves no residue, odorless and colorless.

Provides rapid disinfection, 2 to 5 minutes for surfaces, and 20 minutes for submerging surgical instruments.

4 AREAS OF USE

Healthcare sector: Hospitals, operating rooms, intensive care units, isolation rooms, healthcare centers, dental clinics, medical and laboratory equipment, cosmetic clinics, cupping centers, and laboratories.

Public facilities: Crowded areas, Haj and Umrah sites, masjid, hotels, educational institutions, and sports facilities.

Transportation: Aircraft, trains, buses, ambulances, taxis, and vehicles.

Industrial sector: Factories, warehouses, and infrastructure facilities.

5 DIRECTIONS FOR USE

1. Spray Technique

The bottle nozzle is designed to provide both wide spray coverage and direct targeted spraying. Shake well before use, spray onto clean surfaces, and leave for 2 to 5 minutes before final wiping.

2. Fogging Technique

Suitable for use with fogging machines. The fine particles generated are safe for medical equipment, making it ideal for sensitive healthcare environments.

3. Submerging Technique

Submerge surgical instruments for a minimum of 20 minutes to ensure effective disinfection. Important Note: Do not spray directly onto sensitive electronic devices to avoid damage or electrical hazards. It is recommended to spray onto a cloth first, then gently wipe the surfaces.


BROAD-SPECTRUM PROTECTION

Effective against bacteria, viruses, coronaviruses, fungi, and algae.

FAST & EFFECTIVE

Up to **%99.99** effectiveness in minutes.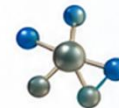

ENHANCED WITH NANO SILVER

Advanced nano-silver technology for superior disinfection.

SAFE & RELIABLE

Advanced formula designed for healthcare environments.

ECOSAN[®] NM-5

With Nano-Ag⁺ Technology

500 mL
Spray Bottle

1 Liter
Standard Bottle

5 Liters
Gallon

10 Liters
Drum

20 Liters
Drum

700 Liters
Palletized Container

LABORATORY TEST RESULTS

✓ Tested against a wide range of microorganisms in accordance with international standards.

STANDARD	MICROORGANISM	STRAIN
EN 14476	Influenza A (H3N2)	ATCC VR-544
	Murine Norovirus	FLU-RVB-0651
EN 13727	Escherichia coli K12	NCTC 10538
	Pseudomonas aeruginosa	ATCC 15442
	Enterococcus hirae	ATCC 10541
	Staphylococcus aureus	ATCC 33592
EN 1656	Proteus vulgaris	ATCC 13315
EN 1276	Salmonella	ATCC 13332
	Listeria monocytogenes	ATCC 13932
	Campylobacter jejuni	ATCC 33291
	Escherichia coli	ATCC 10708

EFFECTIVE PROTECTION

Proven efficacy against bacteria, viruses, fungi, and algae.

BACTERIA

VIRUSES

FUNGI

ALGAE

Tested according to European Standards

ISO CERTIFIED

GMP QUALITY

MEDICAL DEVICE

ECOSAN[®]-5

Multipurpose Disinfection

6% HYDROGEN PEROXIDE

Broad-Spectrum Ecological Disinfectant

MADE IN
SAUDI ARABIA

Contains hydrogen peroxide at a concentration of **6%** to ensure maximum disinfection efficiency.

Broad-Spectrum Efficacy
Against Bacteria, Viruses,
Fungi, Spores, Algae
and Amoebae

Odorless, Colorless,
Non-Foaming

Compatible with
Fogging and
Fumigation Machines

Biodegradable,
Decomposes to
Oxygen and Water

Rapid Surface
Disinfection
in 2-5 Minutes

Free from Alcohol,
Aldehydes, Chlorine,
Phenols and QAC

Instrument Immersion
in 20 Minutes

Non-Carcinogenic
and Eco-Friendly

Thermally Stable
up to 113°C

Proven Efficacy
up to 99.99%

DESCRIPTION:
ECOSAN-5 is a powerful multipurpose disinfectant based on 6% Hydrogen Peroxide. It is effective against a wide range of bacteria, viruses, fungi and spores. Suitable for use on surfaces, instruments and equipment.

DIRECTIONS FOR USE:
Spray directly on the surface or item to be disinfected. Allow to remain wet for at least 1 minute. Wipe if necessary or let it air dry. No rinsing required.

CAUTION:

- For external use only.
- Avoid contact with eyes and open wounds.
- Keep out of reach of children.
- Store in a cool, dry place away from sunlight.

- General disinfection for surfaces, instrument and tools.
- It is used to eliminate pathogens that cause diseases.

- Free of:
- Alcohol
 - Aldehydes
 - Chlorine
 - Phenols
 - QAC

**6%
HYDROGEN
PEROXIDE**

500 ml

- ECO FRIENDLY
- NON FOAMING
- ODORLESS COLORLESS

500 ml 6 291100 981656

TECHNICAL SPECIFICATIONS

Active Ingredient	Hydrogen Peroxide 6% w/w
pH (20°C)	2.0 - 4.0
Specific Density (20°C)	1.013 kg/L
Physical Appearance	Colorless, Odorless Liquid
Shelf Life	2 Years in Closed Packaging

APPLICATIONS

- Hospitals and Healthcare Facilities
- Operating Rooms and ICUs
- Dental Clinics and Laboratories
- Public Facilities and Transportation
- Industrial and Food Processing Areas

AVAILABLE SIZES

STANDARDS & CERTIFICATIONS

Innovating Today for a Safer Tomorrow

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